

NEW

Super-Bond Brush & Dip

The famous self curing 4 meta/TBB resin
for a new indication

Splinting mobile teeth

No metal wire
No fibers
No adhesive

ONLY
Super-Bond
Brush & Dip



The true elastic bonding



By courtesy of Dr Kuntz (Paris)

After 7 years: Super-Bond is the
only resin able to bond and stay
flexible without debonding or
fracturing

A Classic... adapted for the new indication



Clear powder with spherical particles for a better mix
Powder/Liquid



Plastic mixing pad usable at room temperature



Gel type acid etching for enamel

A simple technic !



Soak the brush with mixed Base and Catalyst



When you dip the brush into the powder, make a VERY small circle with the brush tip on the surface of the powder. Use a VERY slow motion and wait for a second to let the liquid and powder blend well at the brush tip.

Form a resin ball



Apply on etched enamel:
let it cure 5 mn



The Super-Bond Brush & Dip Starter Kit contains:

- 1 Catalyst 0.3 ml
- 1 Enamel Etchant gel RED 1 ml + needle tips
- 1 Quick Monomer 3.5 ml
- 1 Brush & Dip Clear powder 1.5 g
- 1 Dispensing Stand + Cups
- 2 Brushes
- 1 instructions booklet
- 1 Working card