Aesculap® Dental Implantology

Competence in Dentistry

Aesculap Dental

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Dentistry is currently facing drastic changes. The current state of medical and technological knowledge means that operations can be performed with deepening precision and using less and less invasive techniques. Along with the availability of increasingly sophisticated diagnostic tools, the decisive factor underlying this progress is the parallel ongoing development of the entire range of dental instruments.

With its new and innovative products, Aesculap is playing a decisive part in this development. Aesculap's products are developed strictly by the close contact to our customers.

This applies as much to new and innovative areas such as micro-surgical instruments for implantology, periodontics and endodontics, as to standard dental instruments in general.

Throughout the world, Aesculap products have proved themselves in day-to-day practice, not least from the point of view of quality and cost. With limited investment budgets available, optimized

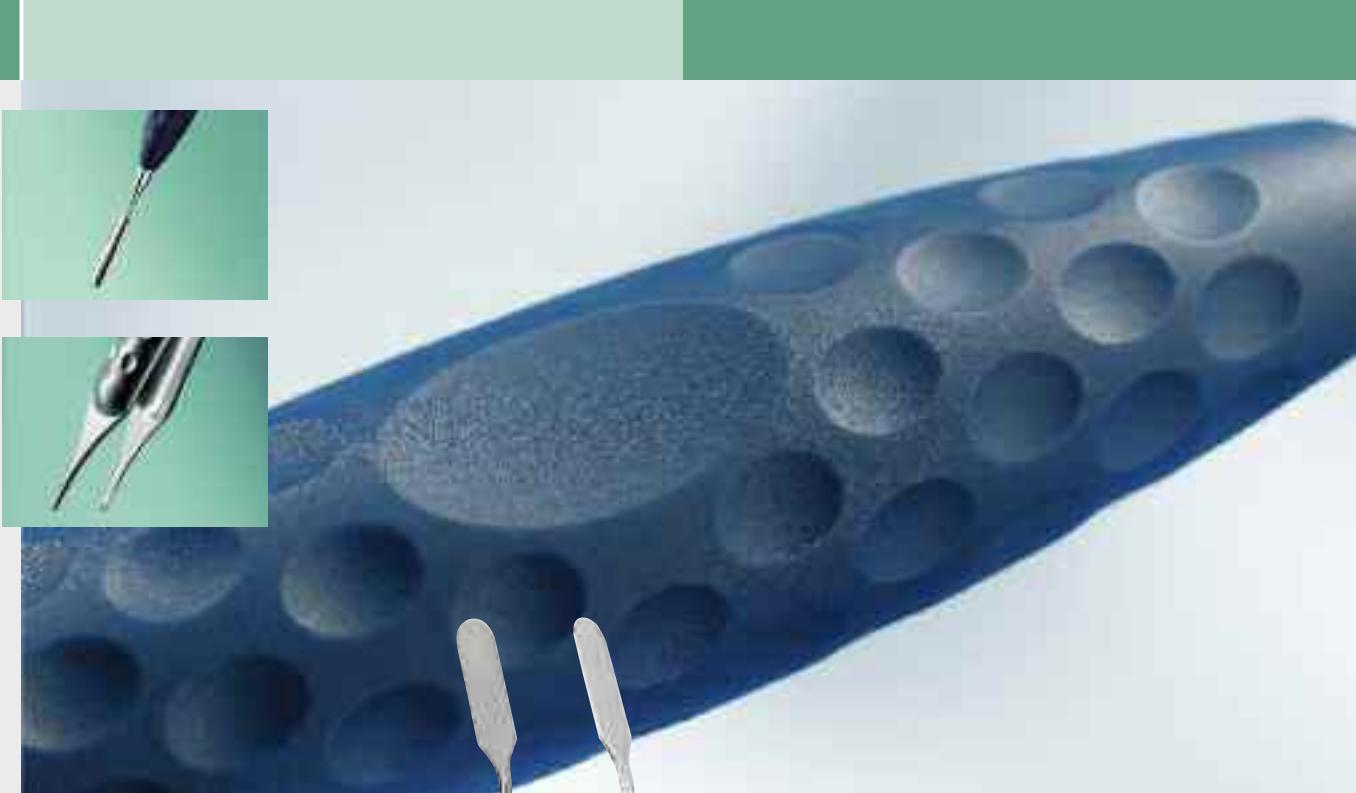
hospital procedures and resulting efficiency gains offer a means of making better use of existing resources. Aesculap's range of services can help to achieve such improvements.

The following service components enable us to meet the ever increasing demands of the future:

- our many years of experience and competence in the field of planning and advice,
- the world renown excellent quality of our products,
- our high level of product availability,
- individual and practice-oriented service concepts to maintain the value of your dental instruments,
- continuous new opportunities for ongoing and further training in the form of the courses offered by the Aesculap Academy.

With ERGOPLANT, the ergonomically designed instrument for dental implantology, Aesculap is setting new standards:

For the first time a complete and unitary product-line, which covers all indications, is available to the implantology focused dentist. Where ever necessary, the working tips had been optimized for each indication. Due to the ergonomic handle design the user is supported by easiness and grip. Finally the surgeon can concentrate on the patient respectively the surgery. The instrument handling is done intuitively.

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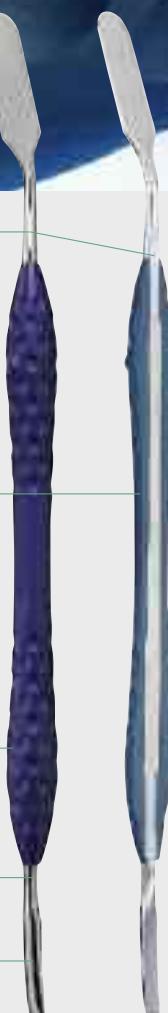
In close collaboration with Dr. Dr. Andres Stricker, Freiburg University, an instrument range especially designed for dental implantology has been developed for optimal ergonomics concerning the working tips and handle for any specific indication.

ERGOPLANT thus enables working efficiently with minimum physical strain under practice conditions.

All ERGOPLANT instruments are suitable for steam sterilization 134°C (Autoclav).

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Implantology Basic Kit

The standard instrumentation for dento-alveolar operations.

DX705

Aesculap Basic Kit for dental implantology, consisting of:

BB515	1 PAK = Package of 100 scalpel blades, sterile, Fig. 15
BB063R	1 Scalpel handle, No. 3
DX094	1 ERGOPLANT Mouth mirror holder with PEEK handle
DA036R	1 PAK = Package of 12 Rhodium-Mouth mirror, 22 mm diam.
DX051R	1 ERGOPLANT Tissue forceps
DX052R	1 ERGOPLANT Dissecting forceps
DX201R	1 ERGOPLANT Raspatory, BUSER
DX200R	1 ERGOPLANT Raspatory, PRICHARD
DX220R	1 ERGOPLANT Bone curette, LUCAS
BM065R	1 DUROGRIP® Needle holder, HEGAR-MAYO
D0219R	1 Gum scissors, LOCKLIN
DX500R	1 ERGOPLANT Bone rongeur, LUER-FRIEDMANN
JG381R	1 Tray (273 x 176 x 30 mm), suitable for dental container JN092

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Implantology Advanced Kit

Special instrumentation designed for the particular requirements of the bony implant site.

With this instrumentation it is possible to remove inflamed soft tissue residue completely while achieving maximum preservation of alveolar bone.

This reduces the need for augmentative measures.

ERGOPLANT – Advanced Bone Preparation

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DX706

Ideal for detailed preparation of the boney implant site and for thorough removal of soft tissue residues. Also suitable for minor augmentative procedures.

Aesculap Advanced Kit for dental implantology, consisting of:

- DX230R 1 ERGOPLANT Excavator
- DX231R 1 ERGOPLANT Excavator
- DX241R 1 ERGOPLANT Universal curette, M23A
- DX300R 1 ERGOPLANT Furcation probe, NABERS
- DX400R 1 ERGOPLANT Periotome
- DX253R 1 ERGOPLANT Bone file, MILLER
- DX252R 1 ERGOPLANT Bone file, MILLER
- DX555R 1 ERGOPLANT Applicator for bone material, STRICKER
- DX600R 1 ERGOPLANT Membrane placing instrument, STRICKER
- DX501R 1 ERGOPLANT Bone cutting forceps
- JG381R 1 Tray (273 x 176 x 30 mm), suitable for dental container JN092

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Filigree-aesthetic soft tissue management in implantological operations as precondition for a good implantation.

These instruments can be used as successful for larger or coarser mucocutaneous incisions as for the sensitive, easily torn mucous membrane in the area of the neighbouring teeth and their papillae. In soft tissue augmentations, such as free mucous membrane or connective tissue transplants, both harvest and transplantation site can be prepared simultaneously in exemplary fashion.

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DX707

Aesculap Soft Tissue Kit for dental implantology, consisting of:

BB045T	1 Titanium scalpel handle
BB367R	1 PAK = Package of 10 micro scalpel blades, sterile, Fig. 367
BB063R	1 Scalpel handle, No. 3
BB515	1 PAK = Package of 100 scalpel blades, sterile, Fig. 15
DX050R	1 ERGOPLANT Tissue forceps
BM003R	1 DUROGRIP® Needle holder
DX450R	1 ERGOPLANT Gingivectomy knife, KIRKLAND
DX203R	1 ERGOPLANT Raspatory
DX202R	1 ERGOPLANT Raspatory, STRICKER
DX405R	1 ERGOPLANT Atraumatic papilla elevator, PHW
DX401R	1 ERGOPLANT Spatula, sharp, STRICKER
DX402R	1 ERGOPLANT Spatula, HEIDEMANN
BC061R	1 Delicate scissors, curved
DX055R	1 ERGOPLANT Soft tissue forceps
JG381R	1 Tray (273 x 176 x 30 mm), suitable for dental container JN092

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In case of poor bone thickness up to 3mm, it's possible to split the jaw-bone (Bone Split) and separate it by means of special chisels, the so called Osteotomes. Finally a gap occurs between buccal and lingual respectively buccal and palatinal bone-lamella, where an insert can be implanted.

Another indication could be the "chiseling out" of bone-transplants. In any case the user is supported by the ERGOPLANT handle design. Together with Dr. Dr. Stricker, we developed a special "Step-Osteotome". Due to this innovative instrument, cracks on the bone-lamella, as often occurs particularly on thin jaw-bones, can be avoided now.

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DX708

Aesculap Bone Split Kit for dental implantology, consisting of:

DX510R 1 ERGOPLANT Osteotome, straight, 3 mm
DX511R 1 ERGOPLANT Osteotome, straight, 4 mm
DX512R 1 ERGOPLANT Osteotome, straight, 6 mm
DX513R 1 ERGOPLANT Osteotome, straight, 8 mm
DX517R 1 ERGOPLANT Osteotome, curved, 4 mm
DX518R 1 ERGOPLANT Osteotome, curved, 6 mm
DX522R 1 ERGOPLANT Step-Osteotome, 6 mm
JG381R 1 Tray (273 x 176 x 30 mm),
suitable for dental container JN092

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Under circumstances the achievement of primary stability in implantations could be problematically. For instance if the upper jaw's bone structure is soft and "comb-like". With the Bone Condenser Kit the bone can be condensed (Bone Condensing) step-by-step until we finally achieve enough primary stability. The aligned conical working tips are convex on the tip. The rather spongy bone is almost "transformed" into compact cortical bone, which now enables a secure and successful implant insertion.

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DX709

Aesculap Bone Condenser Kit for dental implantology, consisting of:

DX530R 1 ERGOPLANT Bone condenser, straight, 1.5 - 2.2 mm
DX531R 1 ERGOPLANT Bone condenser, straight, 2.2 - 2.8 mm
DX532R 1 ERGOPLANT Bone condenser, straight, 2.8 - 3.5 mm
DX533R 1 ERGOPLANT Bone condenser, straight, 3.5 - 4.2 mm
DX534R 1 ERGOPLANT Bone condenser, straight, 4.2 - 4.8 mm
DX535R 1 ERGOPLANT Bone condenser, bayonet, 1.5 - 2.2 mm
DX536R 1 ERGOPLANT Bone condenser, bayonet, 2.2 - 2.8 mm
DX537R 1 ERGOPLANT Bone condenser, bayonet, 2.8 - 3.5 mm
DX538R 1 ERGOPLANT Bone condenser, bayonet, 3.5 - 4.2 mm
DX539R 1 ERGOPLANT Bone condenser, bayonet, 4.2 - 4.8 mm
JG381R 1 Tray (273 x 176 x 30 mm),
suitable for dental container JN092

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In case of upper-jaw augmentation, it is possible to pass the external sinus elevation with its lateral window technique, if still enough vertical bone is available. Then an internal Sinus Elevation by means of the Sinutome Kit can be proceeded. With different sizes of Sinutomes the maxillary sinus floor is elevated through the oral cavity towards cranial. The concave working tip is semi-sharp on the edge, which enables the taking along of additional bone for further elevation. After placing the augmentation-material, the implant can be inserted during the same session.

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DX710

Aesculap Sinutome Kit for dental implantology, consisting of:

DX570R 1 ERGOPLANT Sinutome, straight, Ø 2.2 mm
DX571R 1 ERGOPLANT Sinutome, straight, Ø 2.8 mm
DX572R 1 ERGOPLANT Sinutome, straight, Ø 3.5 mm
DX573R 1 ERGOPLANT Sinutome, straight, Ø 4.2 mm
DX574R 1 ERGOPLANT Sinutome, straight, Ø 4.8 mm
DX575R 1 ERGOPLANT Sinutome, bayonet, Ø 2.2 mm
DX576R 1 ERGOPLANT Sinutome, bayonet, Ø 2.8 mm
DX577R 1 ERGOPLANT Sinutome, bayonet, Ø 3.5 mm
DX578R 1 ERGOPLANT Sinutome, bayonet, Ø 4.2 mm
DX579R 1 ERGOPLANT Sinutome, bayonet, Ø 4.8 mm
JG381R 1 Tray (273 x 176 x 30 mm),
suitable for dental container JN092

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For advanced alveolar crest atrophies less than three millimeters wide it is preferable to pursue a two-stage approach, in which, ideally, an autologous bone graft from appropriate autologous donor sites should be transplanted to the deficient alveolar crest atrophy region prior to the actual implantation.

This is exactly what the Aesculap Bone Fixation Set was developed for.

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Aesthetic

The flat screw head prevents possible soft tissue dehiscence.

Secure

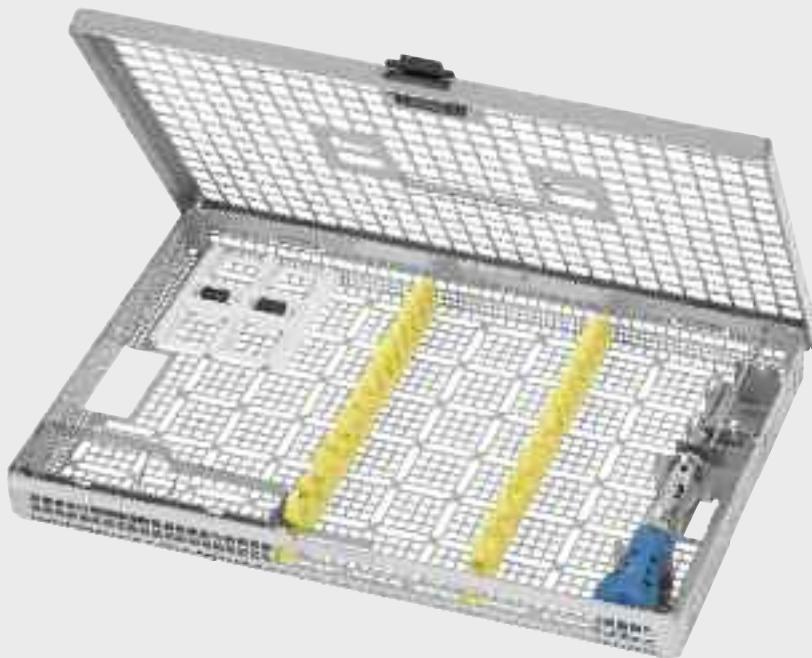
Friction and the screw sleeve ensure secure retention for the screw.

Modular and customisable

Compile your own customised set.

Spotlessly clean

The new open magazine storage system for the screws enables mechanical cleaning of the implants.

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DX715

Aesculap Bone Fixation Set,
consisting of:

- DX605R Screw driver handle
- DX606R Screw driver blade
- DX604R Screw securing sleeve
- DX630R Twist drill, diam. 1.25 mm, 31/16 mm
- DX631R Twist drill, diam. 1.5 mm, 27/12 mm
- DX620 Carrier for 4 screw magazines
- JG403R Tray incl. silicone bars and racks for drills
and screw driver
suitable for dental container JN092

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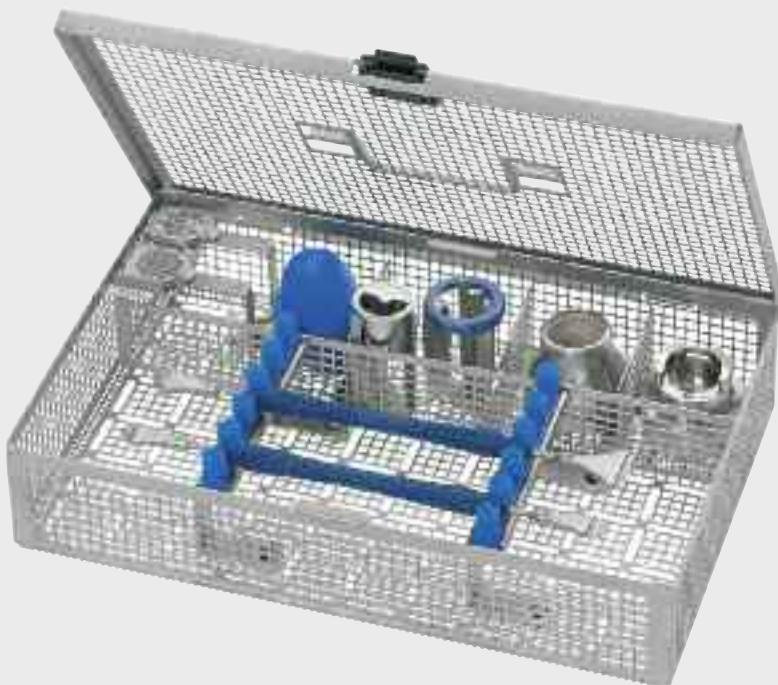
Autogenous bone in augmentation has been established as the "golden standard". The region where to gain the bone (Bone Harvesting) and the required consistence for augmentation can vary, depending on the case. Here the ERGOPLANT Micro Bone Mill shows its flexibility whereat the maximum yield of bone-grist is always ensured.

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DX800

Aesculap Micro Bone Mill Kit,
consisting of:

- DX801R 1 ERGOPLANT Micro bone-mill
(incl. DX803R cutting disk, coarse)
- DX802R 1 ERGOPLANT Cutting disk, fine
- DX810R 1 ERGOPLANT Bone mill-cleaner, fine
- DX811R 1 ERGOPLANT Bone mill-cleaner, coarse
- JF284R 1 Tray (274 x 172 x 60 mm)
with storage aid for micro bone-mill,
suitable for dental container JN095
- TE888 1 Packing stencil (only available for DX800)

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
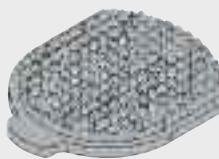
ERGOPLANT
Micro bone-mill
(delivered with coarse
cutting disk DX803R)

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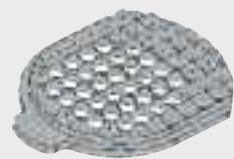ERGOPLANT Micro bone-mill

- small and handy
- interchangeable cutting disks
- cutting disks available for finer or coarser bone particles
- easy to clean due to special cleaning instruments
- easy mounting and demounting
- maximum bone yield requires less bone harvesting

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Recommended accessories for the micro bone-mill:
Cleaning instruments DX810R and DX811R as shown on page 22.

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Recommended Accessories for the ERGOPLANT Micro bone-mill

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Cleaning spatula for radial bone material collecting by cleaning the cutting disk's edge (DX811R the complete surface) at the same time. Valuable bone material can be used for augmentation.

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Cleaning fork for punching out valuable bone material of all cutting disk holes at once in a row.

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Enables secure carrying and placing of enough bone material. The other working tip serves for pushing and plugging bone material into the augmentation area.

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
ERGOPLANT
Applicator for
bone material

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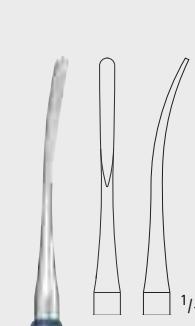The small surface of the pointed tip avoids the sticking of the humid membrane. The small spatula on the other end, helps to take out bone material of, for instance, the bone files DX252R-DX253R, as shown on page 25.

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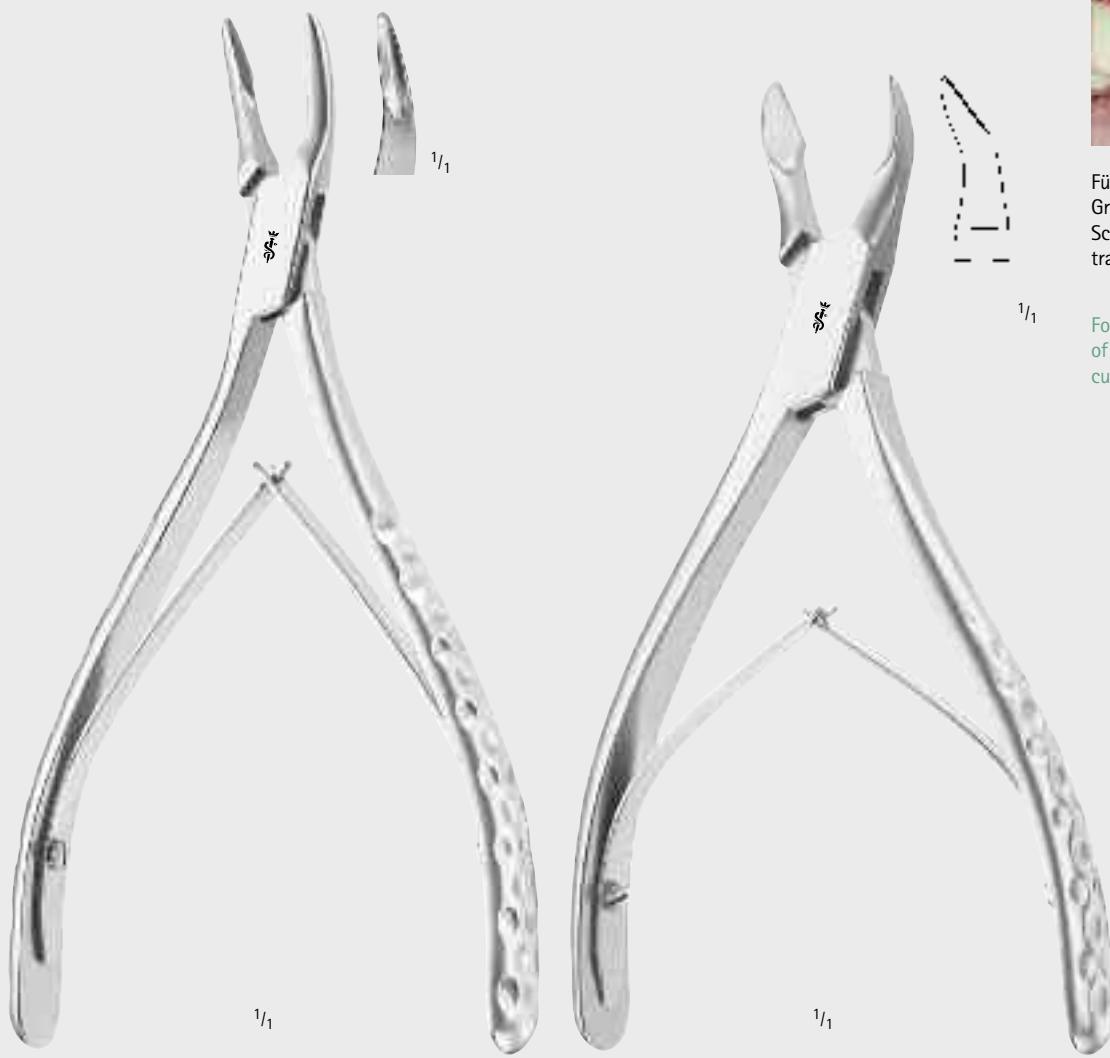
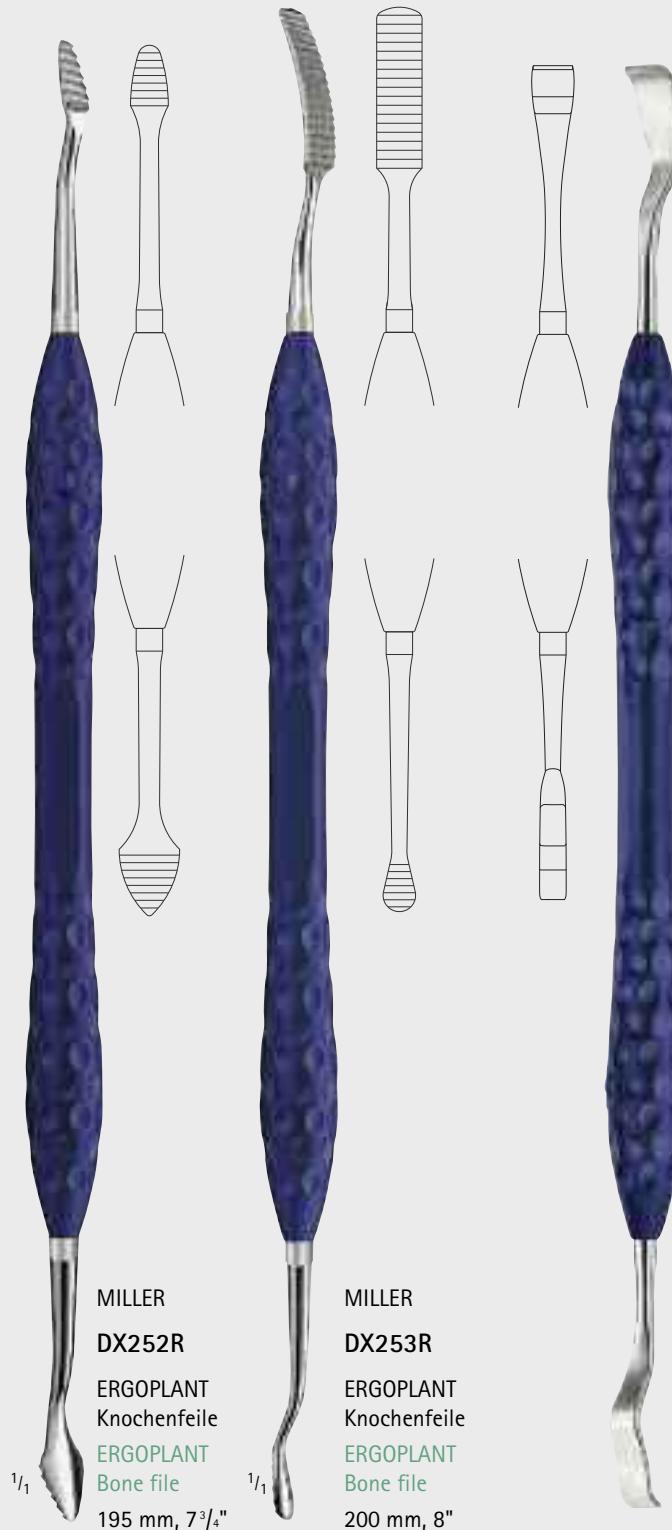
For detailed grabbing and removing of granulated tissue. Also suitable for cutting smaller bone transplants.

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If smaller quantity of bone material is required, with the bone files DX252R-DX253R it can be harvested directly from the jaw-bone, without additional trauma for the patient. The quantity is often sufficient, for augmentation around the implant-neck.

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With this raspatory fine bone particles can be harvested by simply scratching over the jaw-bone.

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Titanium screws

- Highest stability
- Easy removal of residues
- Smallest possible screw head
- Biocompatible
- Secure retention
- Spotlessly clean

FM960T

Titanplatte,
2-Loch,
gerade
Titanium plate
2 holes,
straight

FM964T

Titanplatte,
5-Loch,
Y-Form
Titanium plate
5 holes,
Y-shape

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Screw magazine with 8 titanium screws,
diam. 1.5 mm, self-tapping, cross recess

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Rack for 4 screw magazines
DX608T - DX614T
(delivery without screw magazines)

FM903R

Plattenschneide-
zange

Plate cutter

Schraubendreher

- **Sicher**
Friction and the screw sleeve ensure secure retention for the screw.
- **Ergonomisch**
The ergonomic handle design reduces the force required for tightening or loosening the screw.

Screw driver

- **Safe**
Friction and the screw sleeve ensure secure retention for the screw.
- **Ergonomic**
The ergonomic handle design reduces the force required for tightening or loosening the screw.

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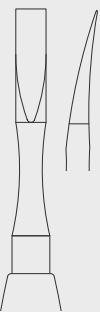
Rack for twist drills and tools
(delivery without drills and tools)

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Osteotome, curved,
4 mm wide,
for bone grafting

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Bone fixation forceps, to fix the bone block at the jaw and to recontour the bone block

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Bone fixation forceps acc. to STRICKER

Adaptable

The working tips adapt to the respective anatomical situation.

Delicate

The delicate working tips enable drilling and screwing even in smaller bone blocks.

Universal

The shape of the forceps allows for universal application in the upper and lower jaw

ERGOPLANT Immediate Implant Retractor

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The ERGOPLANT Immediate Implant Retractor ensures a contamination free and save implantation by holding back the alveolar soft tissue.

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Safe implantation without contamination!

In case of immediate implantation there is a great risk to contaminate the jaw-bone with connective tissue fibre of the soft tissue. The ERGOPLANT Immediate Implant Retractor ensures a save implantation by holding back the alveolar soft tissue. Thereby working will be safer, easier and more comfortable.

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
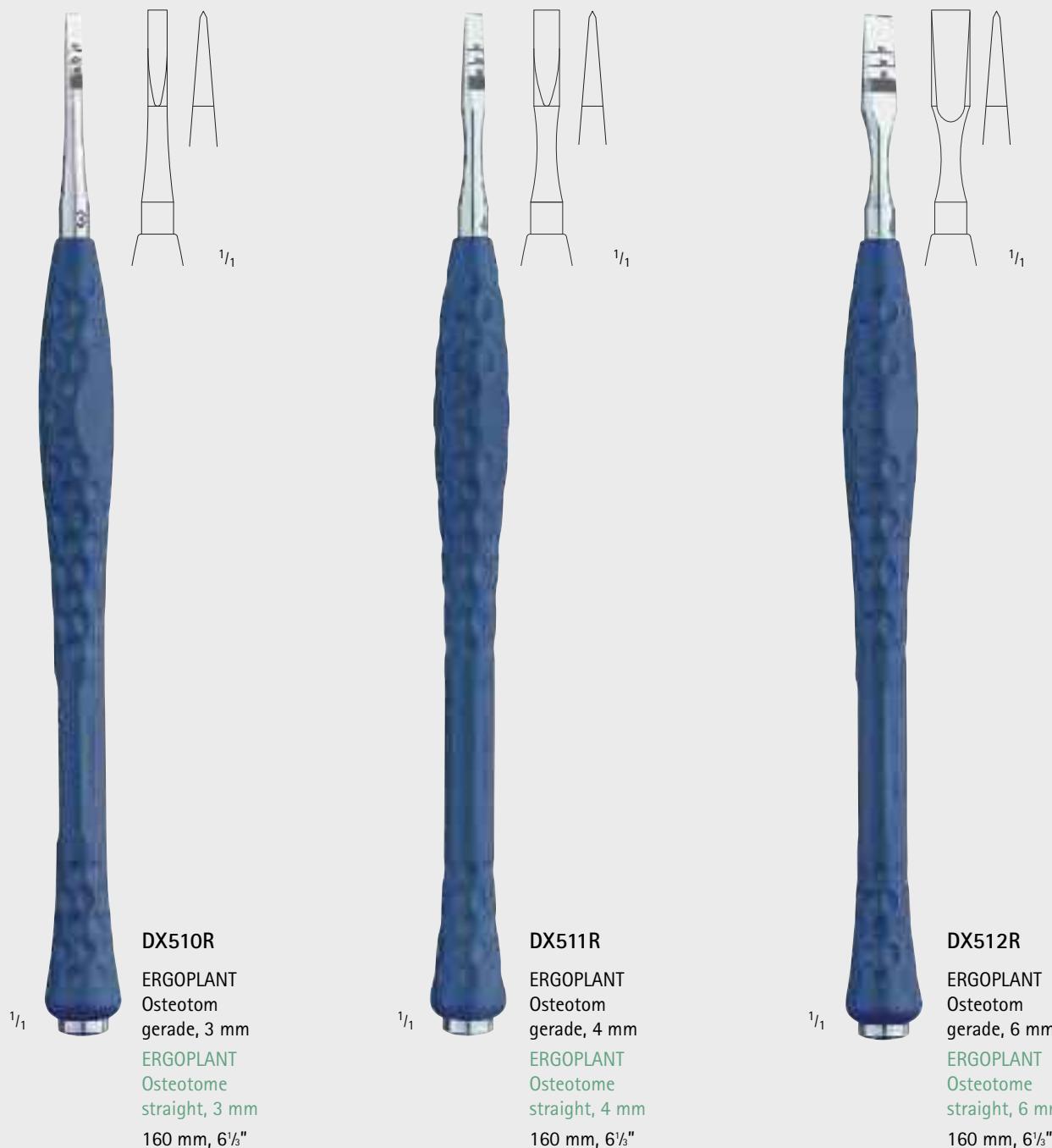
The Step-Osteotome was specially developed for secure bone-splitting with simultaneous stepwise widening. Due to the specially designed working tips, the maxillary crest will be first expanded towards palatinal and only afterwards towards vestibular. In the same time spongy bone in the center will be condensed. After removing the Step-Osteotome, a pre-selection point for the implant bur can be clearly seen.

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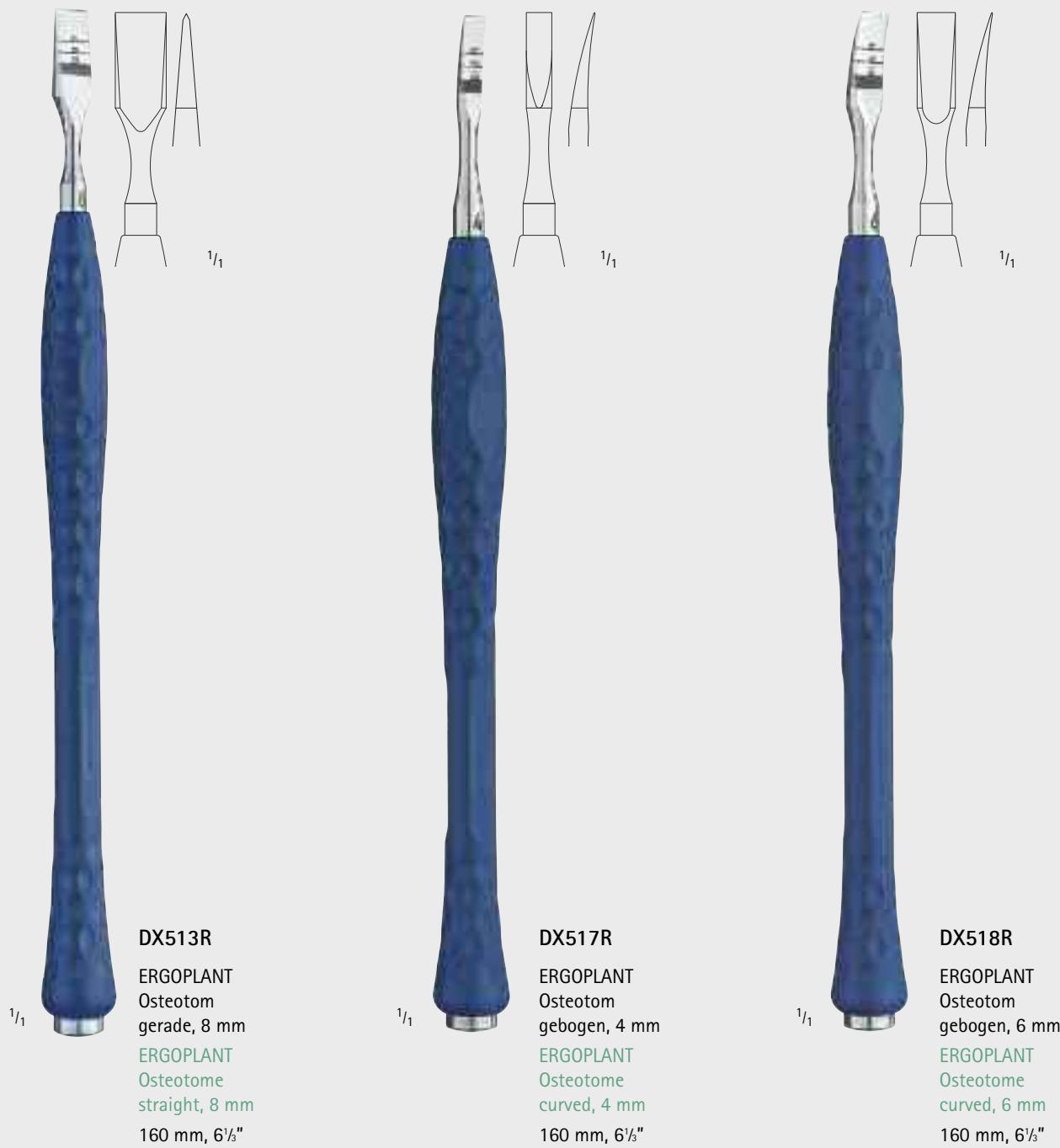The maxillary crest is splitted transversal by the Step-Osteotome.

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The bone split is kept apart by the retractor.

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After splitting the maxillary crest the bone split is closing within a short period of time. This leads to an unwished and unnecessary bone loss when drilling afterwards. This causes the risk of an implant that is not ideally positioned and additional augmentative treatments could be necessary.

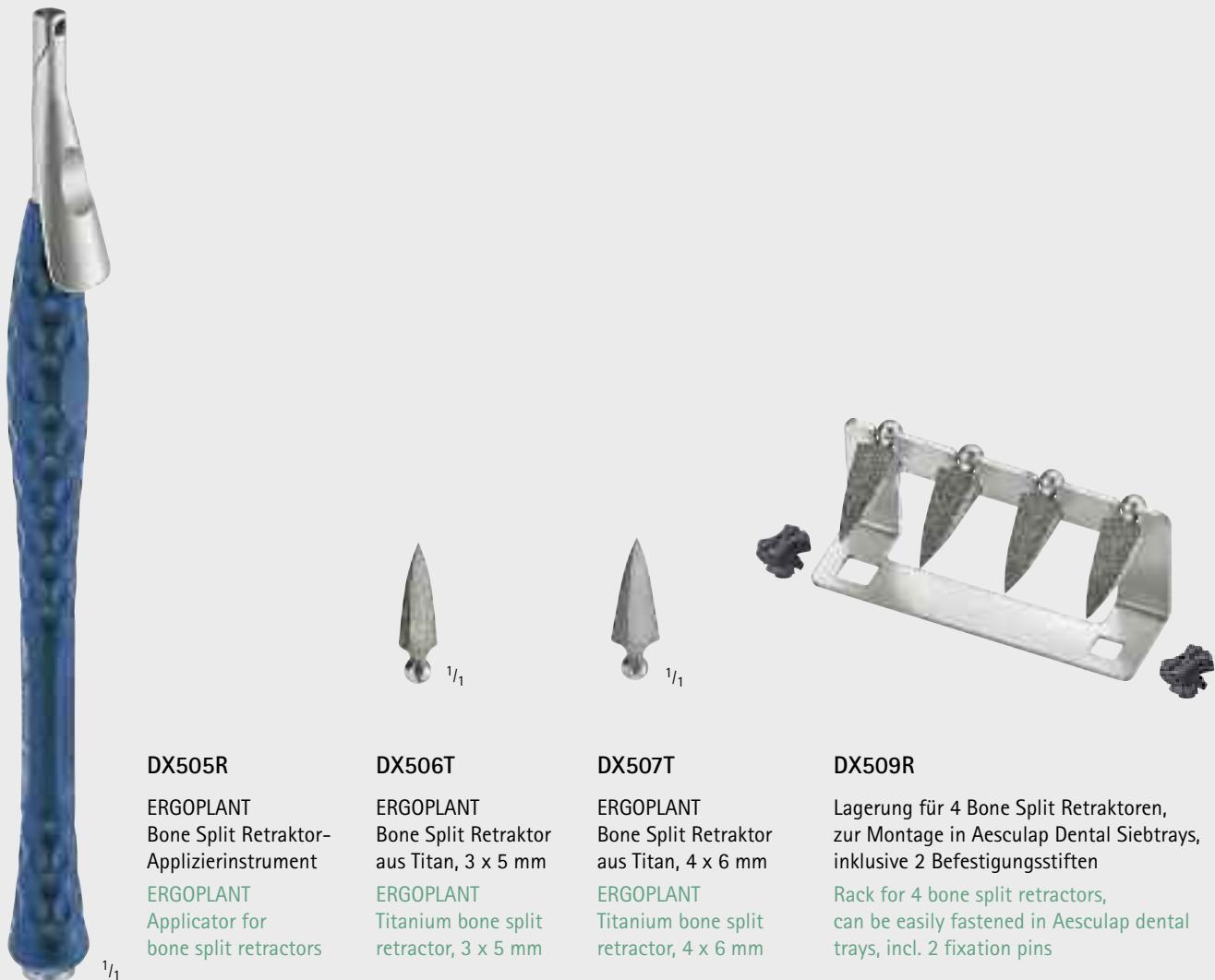
The bone split is kept apart by the retractor. Thus, the existing bone can be used optimally to position the implant.

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The implant can be positioned optimally by using the existing bone.

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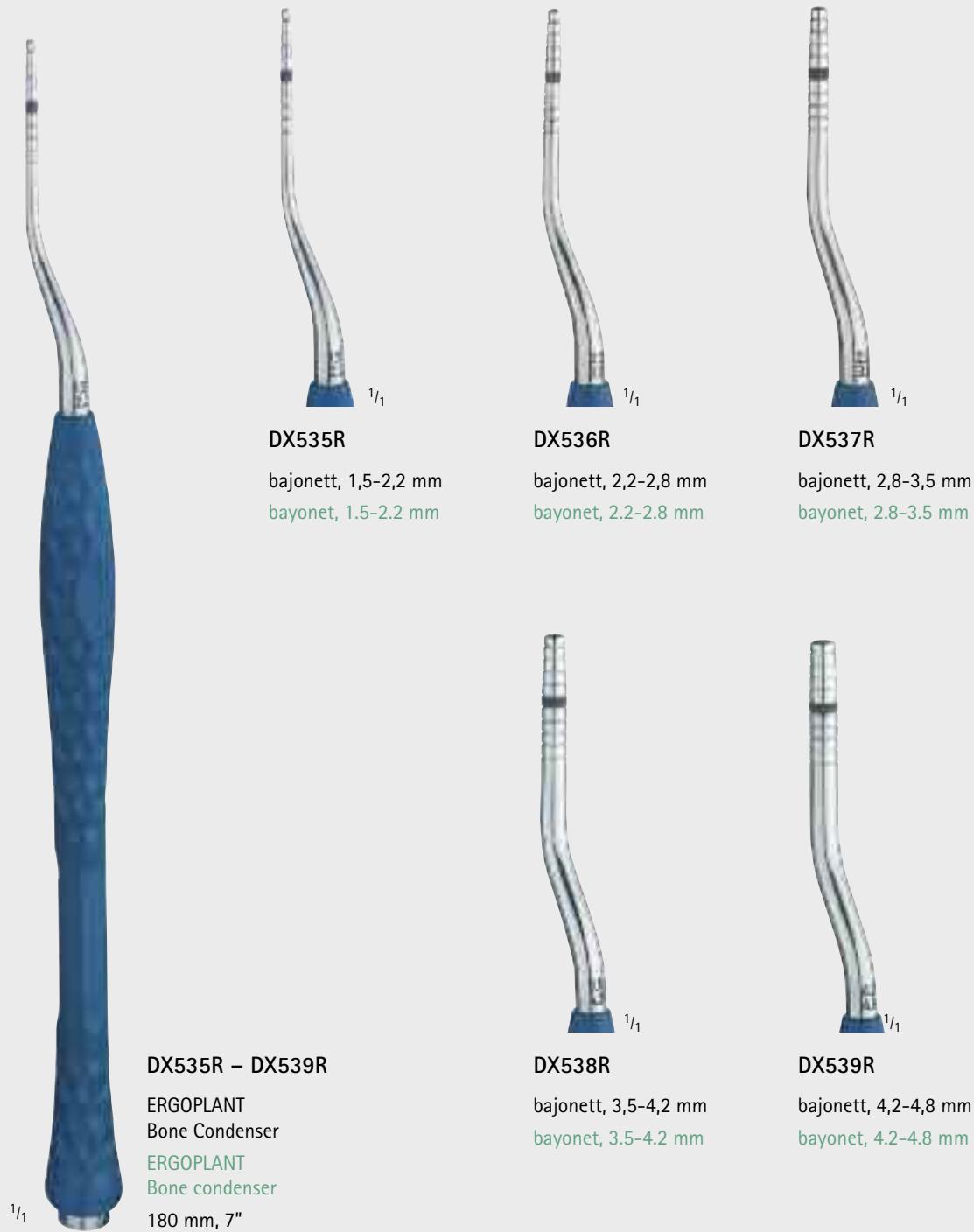
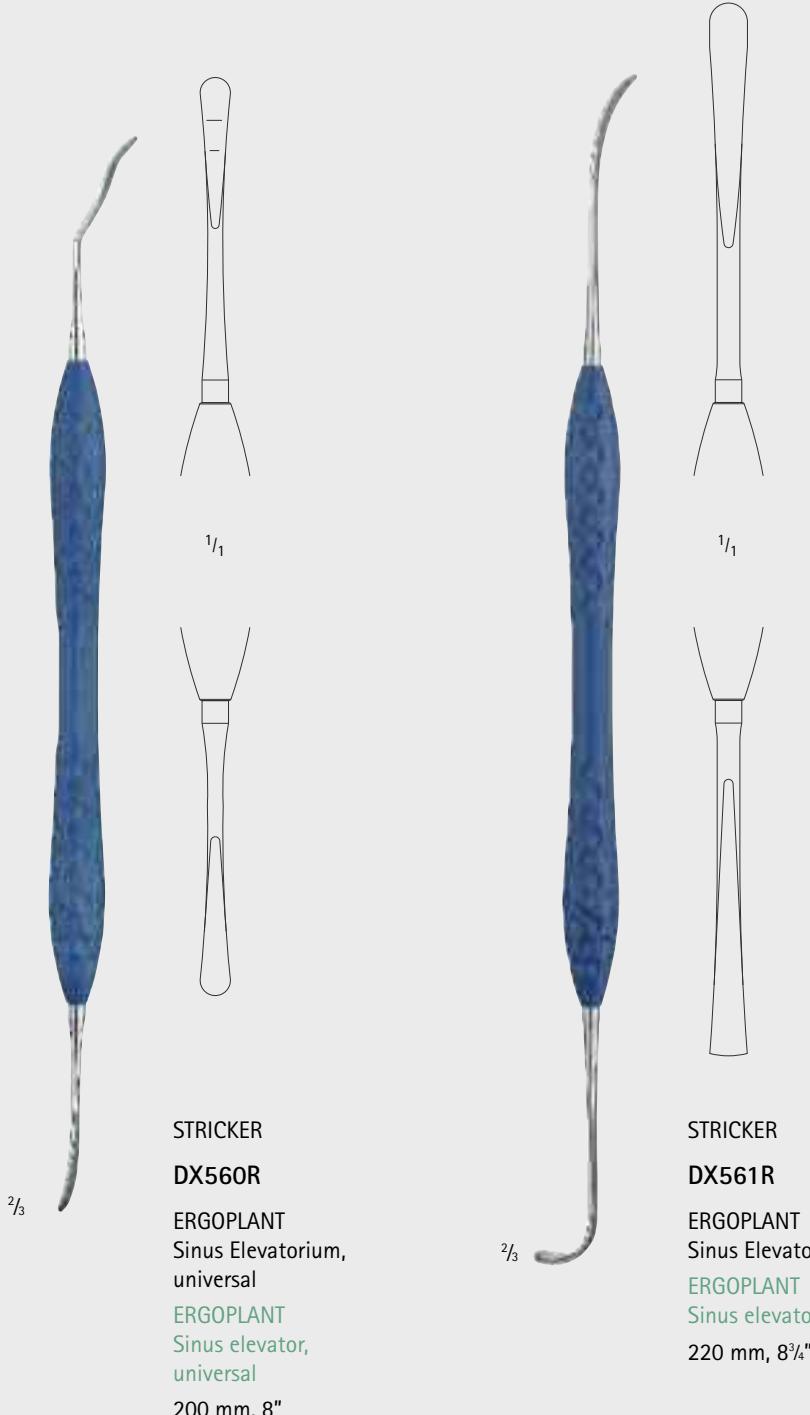
Separating the maxillary sinus membrane carefully from the bony base. Finally the sinus window will be pushed with the other working tip gently into the hollow maxillary cave.

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The Schneiderian membrane of the sinus can be detached with elevator D0571R by 1- 2 mm along the total cortical bone

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Elevator D0573R is particularly useful in the lower part for detaching the membrane further. The same goes for the caudal area, especially when the membrane is 4-5 mm above the base of the maxillary sinus and the maxillary sinus is built up with granular material.

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Elevator D0575R makes it possible to lift the membrane further in mesial region. One working tip is specially designed for the right maxillary sinus and the other for the left maxillary sinus.

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With the aid of elevator D0572R, the sinus membrane is further detached over a broad area in the inner region, especially along the roof of the palate. Condenser D0576R permits positioning of the filling material.

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With elevator D0574R, the sinus membrane is further elevated in the upper and posterior part of the maxillary sinus wall.

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Reduced sinus lift instrument set.

Two double-ended sinus lift instruments consisting of D0576R (5 mm condenser/spoon elevator) and D0577R (90° dissector /raspator).

Many years of experience with sinus lift instruments has shown that this reduced instrument set is more than sufficient for surgical procedures.

The reduced set appreciably lowers the investment the hospital or clinic needs to make.

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Mouth mirror, Rhodium

Specially recommended for dental implantology

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
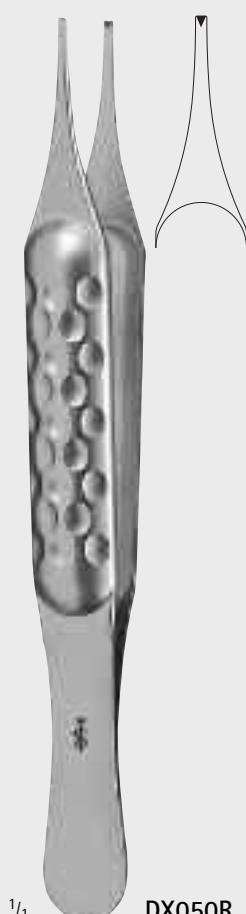
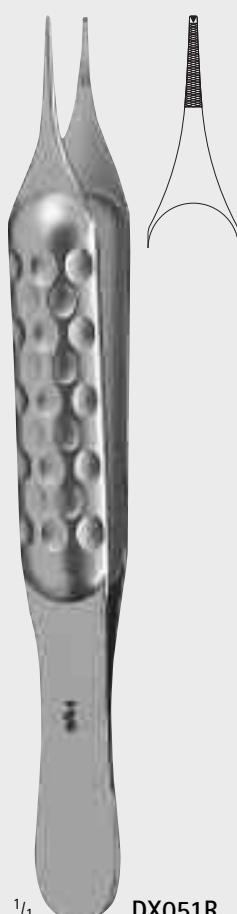
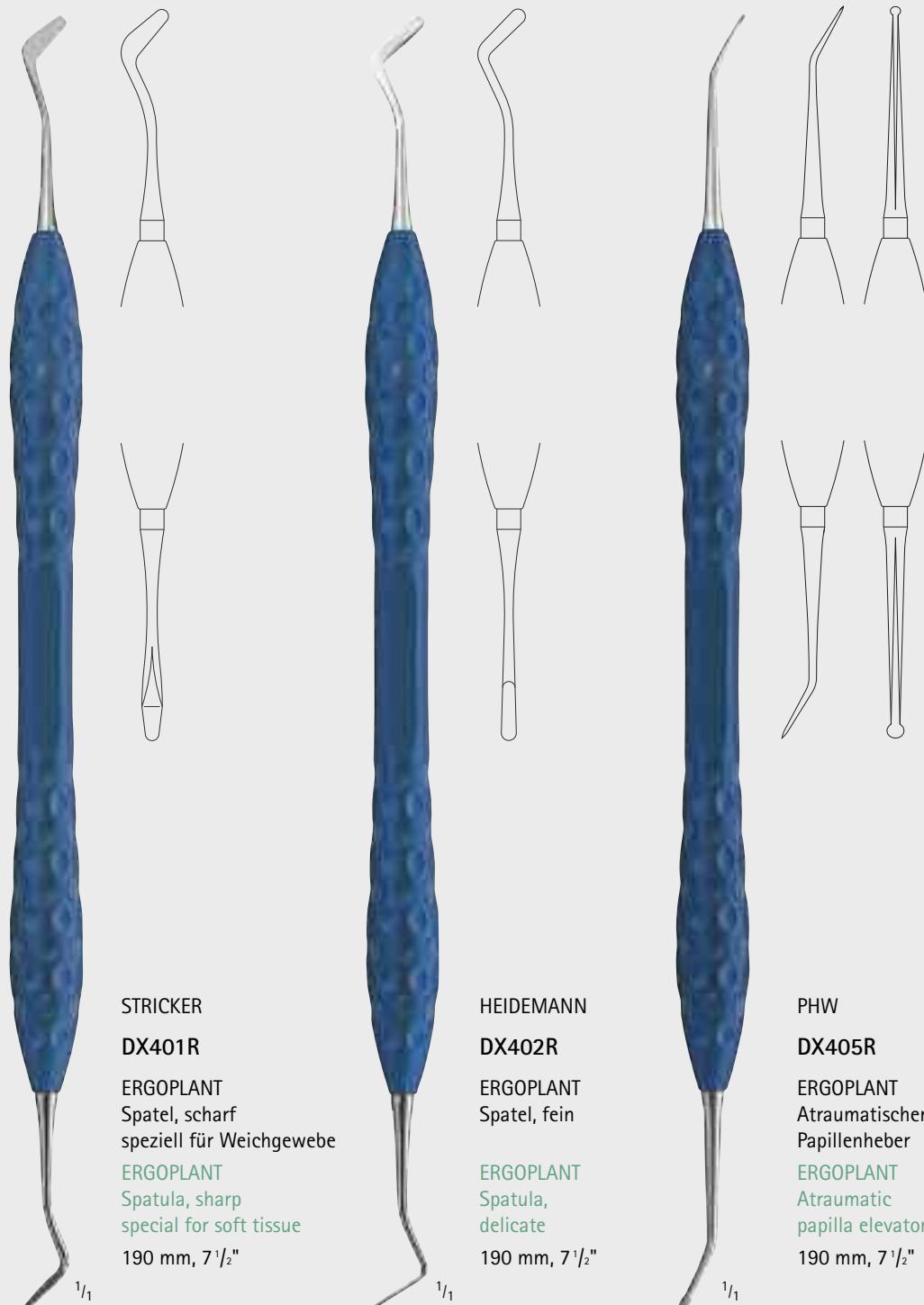
Atraumatic dissecting of the flap by means of the papilla elevator without any damage to the papillae.

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
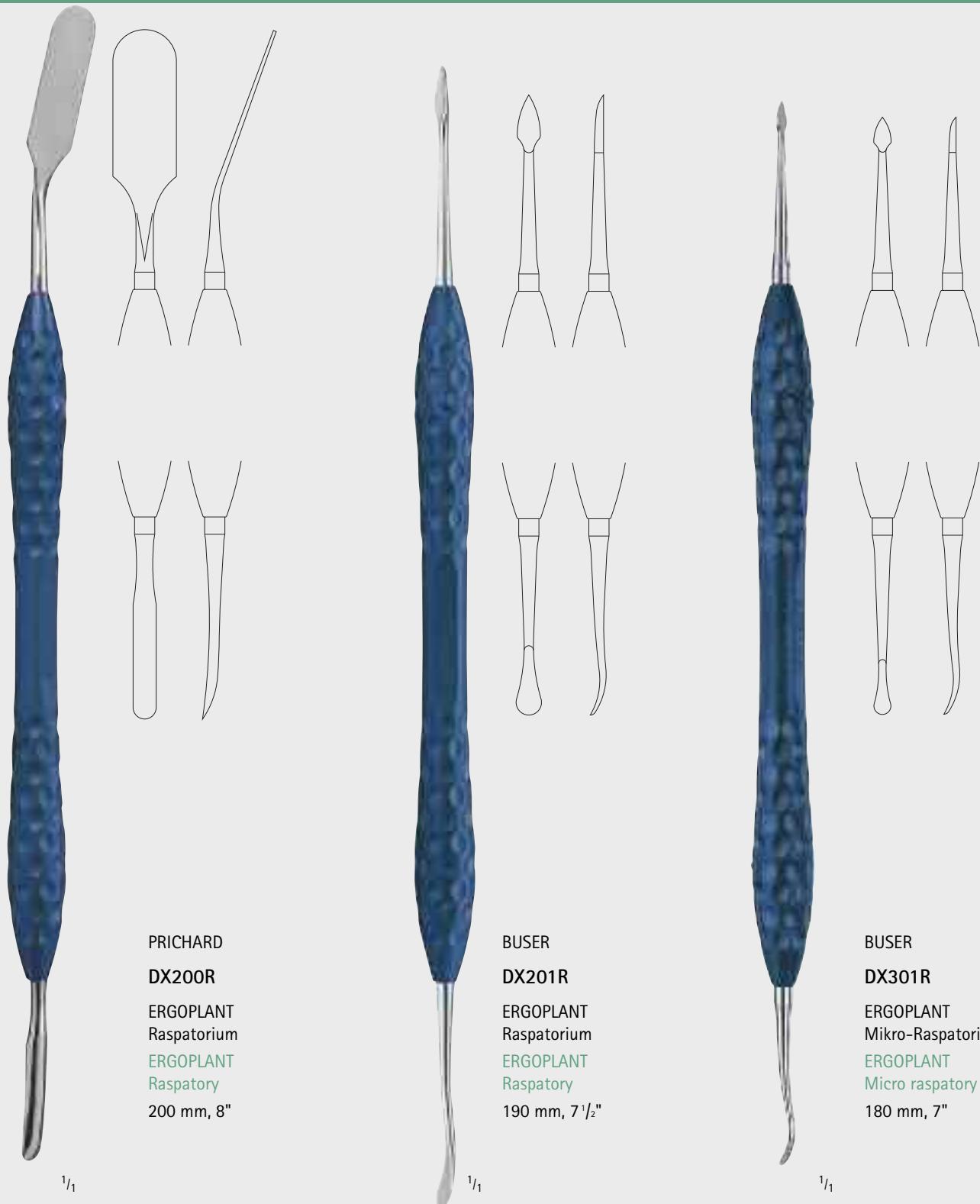
DX720

Micro Soft Tissue Set,
consisting of:

- DX305R Handle for micro blades, 150 mm
- DX301R Micro raspatory, 180 mm
- DX303R Micro tissue forceps, 180 mm
- FD252R Micro ligature scissors, curved,
one blade serrated, 180 mm
- FD253R Micro dissecting forceps, 180 mm
- BM003R DUROGRIP® Needle holder, 180 mm
- JG381R Tray, 274 x 172 x 30 mm

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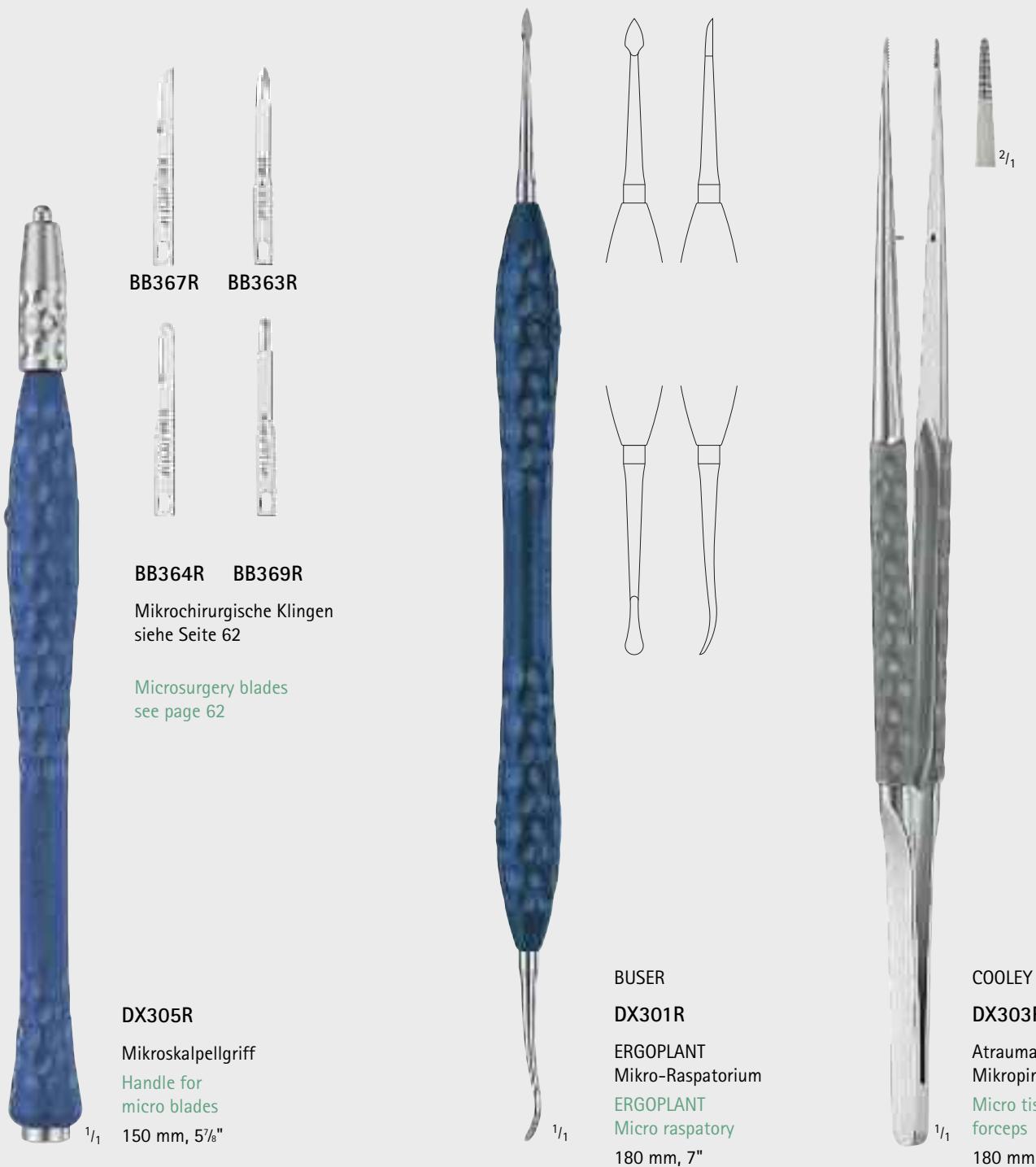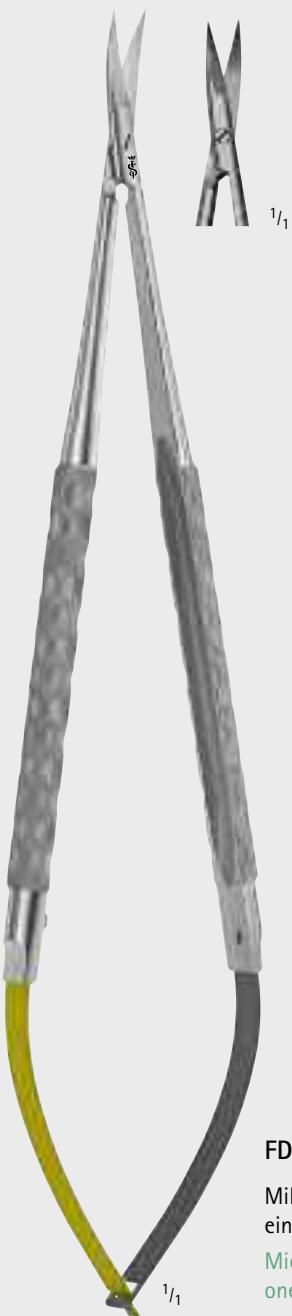
Micro ligature scissors, curved,
one blade serrated

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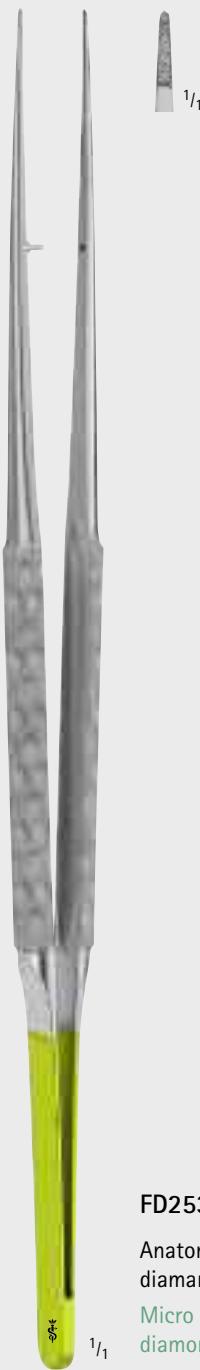
Micro dissecting forceps,
diamond dust coated

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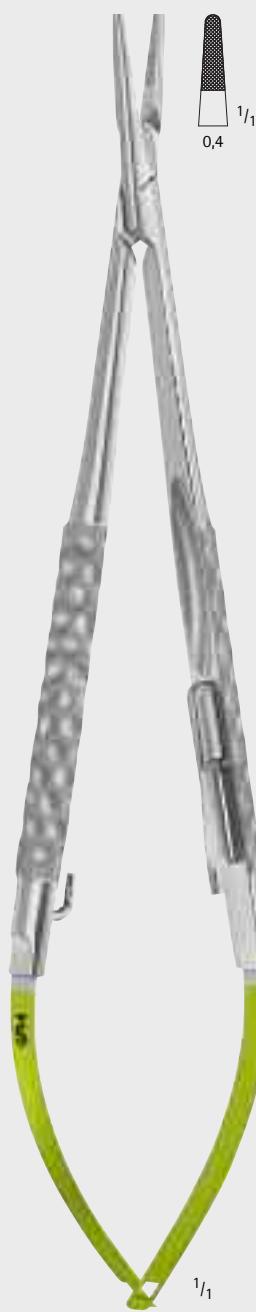This working tip helps
in case of suturing
mucosa transplants,
or as raspatory for
larger surfaces.

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ERGOPLANT
Raspatory for fine soft
tissue

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
ERGOPLANT
Raspatory for soft tissue
sharp/blunt

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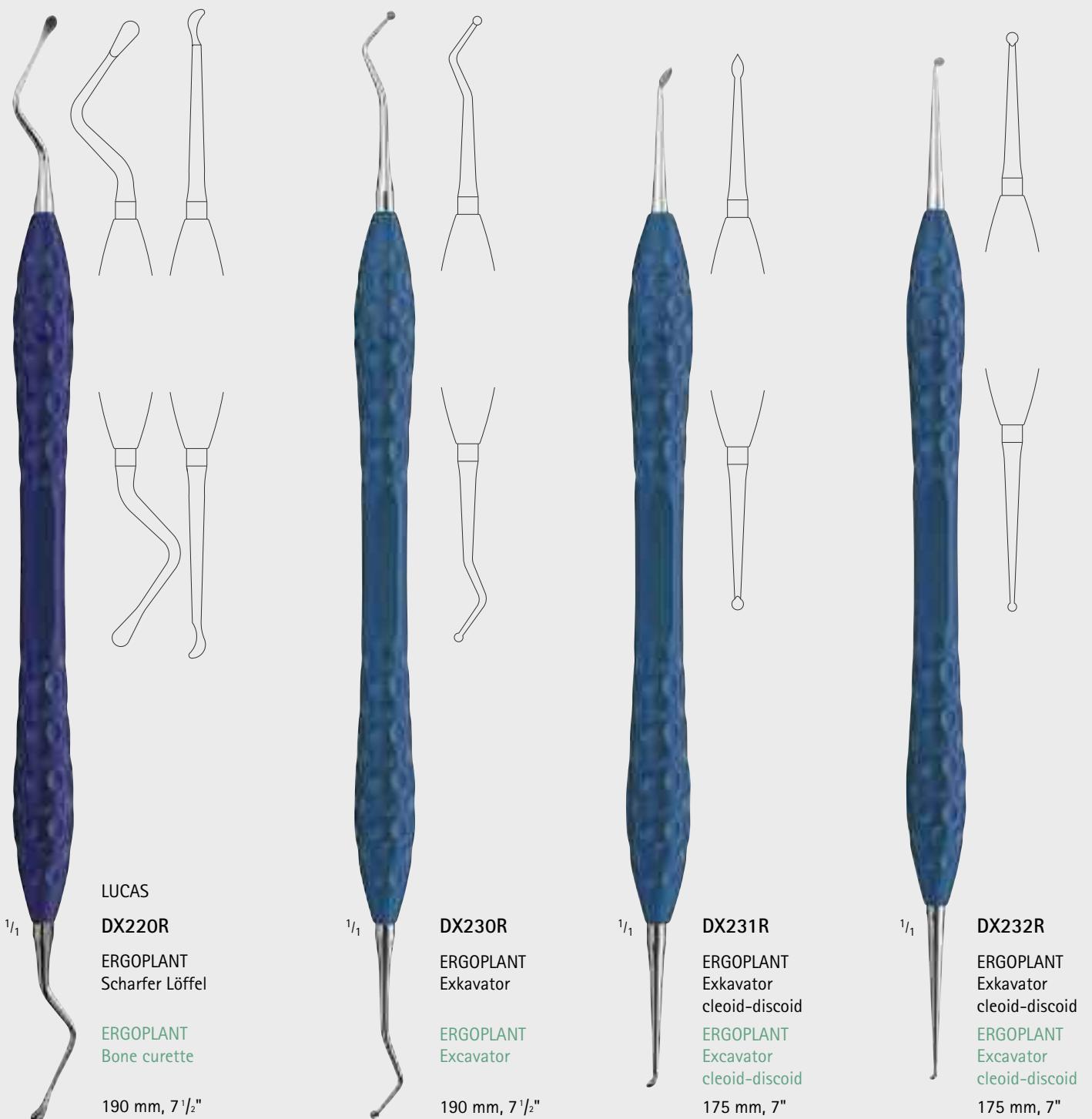
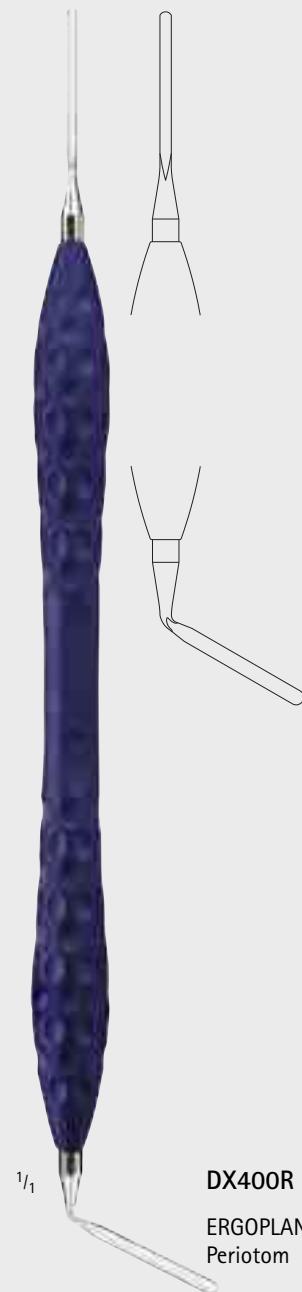
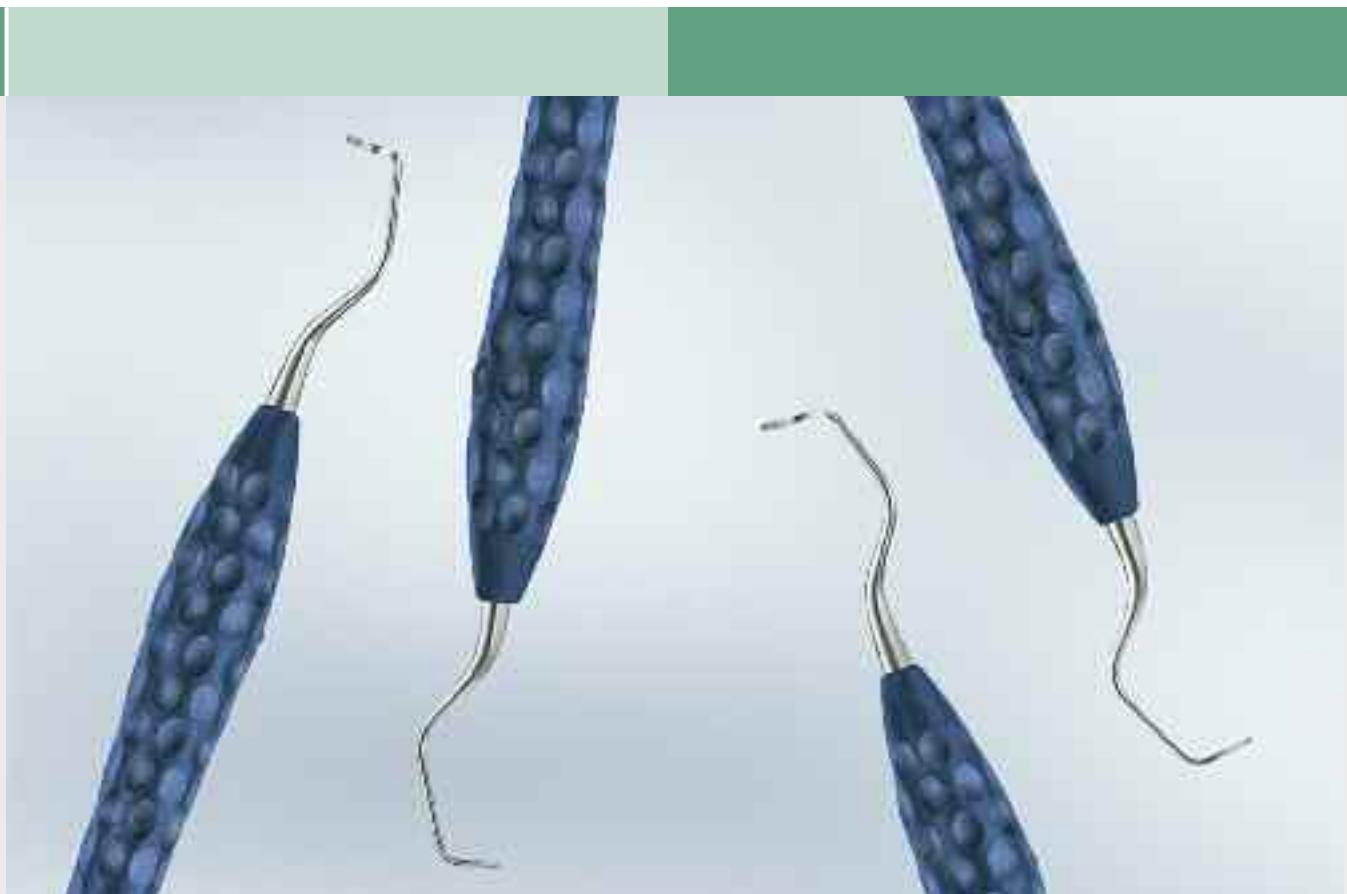
To ensure a successful long-term implantation, a consequent post-operative treatment is very important. To eliminate plaque on the implant – abutment – interface without damaging the implant titanium curettes are the optimal choice.

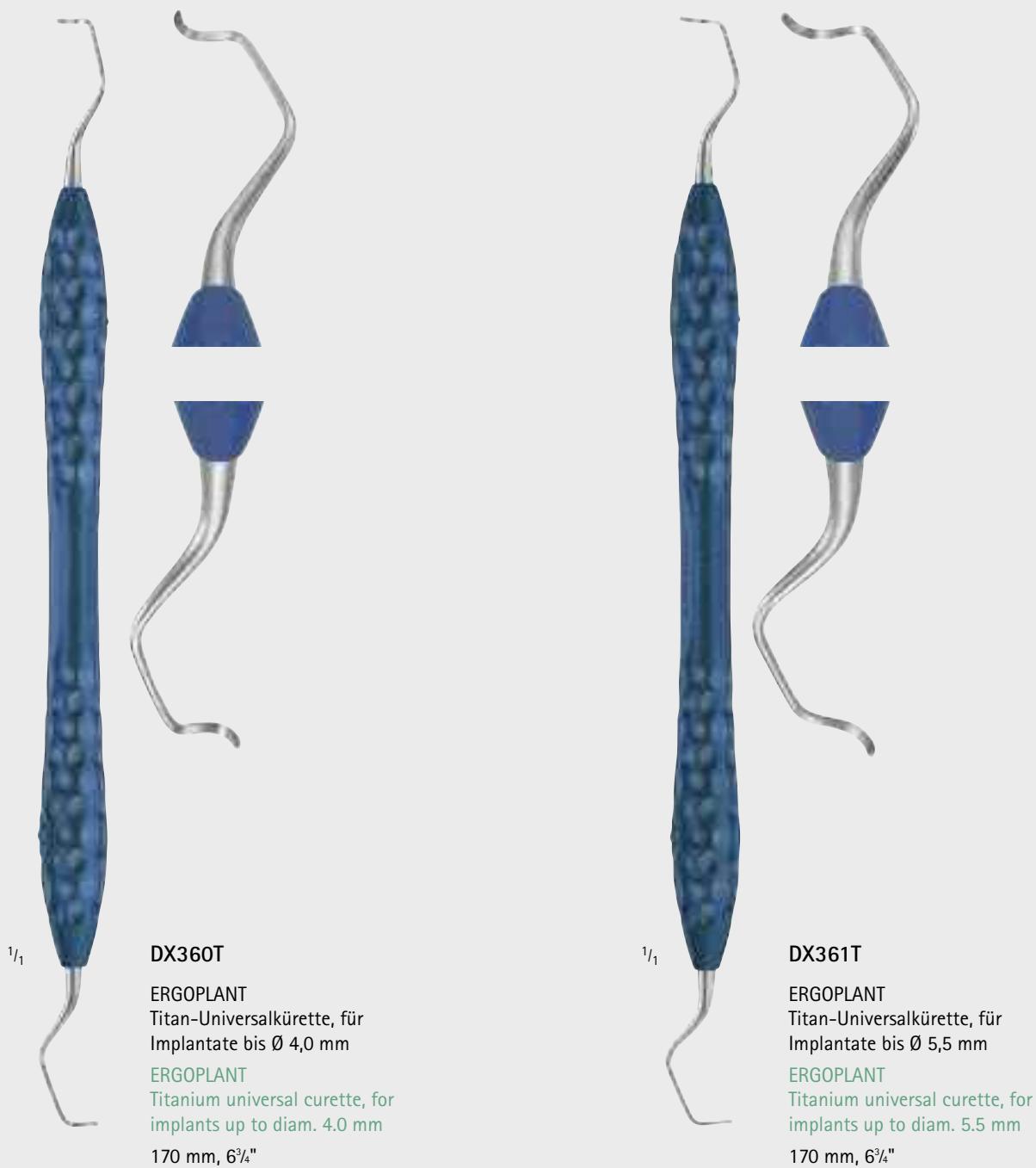ERGOPLANT – Recall Titanium Curettes

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These instruments were developed to hold back lips, cheeks, tongue and lingual mucous membrane. Due to the ergonomic handle design, a more relaxed and fatigue-free working is possible, even in long operations.

DX100R

ERGOPLANT
Wangenhalter

ERGOPLANT
Cheek retractor

215 mm, 8½"

ERGOPLANT – Retractors


DX101R

ERGOPLANT
Vestibulumhaken

ERGOPLANT
Vestibulum hook

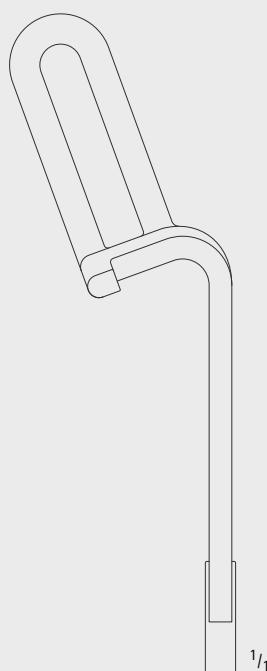220 mm, 8^{3/4}"

BÜCHS

DX102R

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ERGOPLANT
Soft tissue retractor

200 mm, 8"

1/1

1/1

2/3

2/3

DX105R

ERGOPLANT
Zungenretraktor

ERGOPLANT
Tongue retractor

190 mm, 7 1/2"

DX106

ERGOPLANT
Zungenspatel PEEK

ERGOPLANT
Tongue spatula PEEK

190 mm, 7 1/2"

ERGOPLANT – Retractors, Mouth Props, Tongue Retractors

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The mouth wedges are particularly advantageous for longer implantations and also provide support for the patient.

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Sales unit: PAK = Package of 2 pieces
suitable for steam sterilization 134°C
(Autoclav)

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ERGOPLANT
Silicone mouth wedge
Size: S

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ERGOPLANT
Silicone mouth wedge with metal core
Size: M

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ERGOPLANT
Silicone mouth wedge with metal core
Size: L

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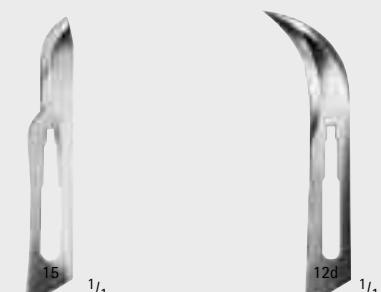
HALSTED-MOSQUITO

BH111R
Arterienklemme
Haemostatic forceps
125 mm, 5"

Sterile Goods Cycle – Instrument Reprocessing

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All hygienic measures in the dental practice are primarily for keeping healthy the patients, the dentist and the assistants.

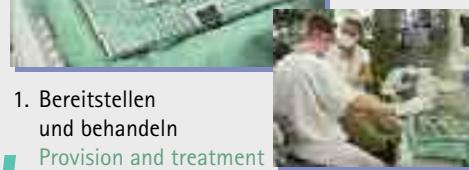
It is particularly important to observe the principles of hygiene when reprocessing instruments.

To keep practice work safe, efficient and economical, clear procedures must be specified for practice hygiene.

The Aesculap Sterile Container System has proven itself as an excellent aid for organizing the instrument reprocessing procedure.

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
Art.Nr. Cat.No.	Seite Page	Art.Nr. Cat.No.	Seite Page	Art.Nr. Cat.No.	Seite Page	Art.Nr. Cat.No.	Seite Page
BB045T	9, 62	DX106	60	DX513R	11, 33	DX630R	17, 27
BB063R	5, 9, 63	DX110R	30	DX517R	11, 28, 33	DX631R	17, 27
BB363R	49, 62	DX113	61	DX518R	11, 33	DX705	5
BB364R	49, 62	DX114	61	DX522R	11, 31	DX706	7
BB367R	9, 49, 62	DX115	61	DX530R	13, 36	DX707	9
BB369R	49, 62	DX120R	61	DX531R	13, 36	DX708	11
BB511	63	DX200R	5, 47	DX532R	13, 36	DX709	13
BB512	63	DX201R	5, 47	DX533R	13, 36	DX710	15
BB515	5, 9, 63	DX202R	9, 52	DX534R	13, 36	DX715	17
BB515-C	63	DX203R	9, 52	DX535R	13, 37	DX720	48
BB542	63	DX220R	5, 53	DX536R	13, 37	DX800	19
		DX230R	7, 53	DX537R	13, 37	DX801R	19, 21
BC061R	9, 64	DX231R	7, 53	DX538R	13, 37	DX802R	19, 21, 22
		DX232R	53	DX539R	13, 37	DX803R	19, 21, 22
BH111R	64	DX240R	54	DX545R	31	DX810R	19, 22
		DX241R	7, 54	DX546	31	DX811R	19, 22
BM003R	9, 48, 51, 65	DX250R	55	DX555R	7, 23		
BM065R	5, 65	DX252R	7, 25, 55	DX560R	38	FD252R	48, 50
		DX253R	7, 25, 55	DX561R	38	FD253R	48, 50
DA036R	5, 44	DX254R	25	DX570R	15, 42		
DA037R	44	DX300R	7, 55	DX571R	15, 42	FM903R	26
		DX301R	47, 48, 49	DX572R	15, 42	FM960T	26
D0219R	5, 64	DX303R	48, 49	DX573R	15, 42	FM964T	26
D0571R	39	DX305R	48, 49, 62	DX574R	15, 42	FM965T	26
D0572R	41	DX360T	57	DX575R	15, 43		
D0573R	39	DX361T	57	DX576R	15, 43	GD580R	63
D0574R	40	DX400R	7, 54	DX577R	15, 43	GD584R	63
D0575R	40	DX401R	9, 46	DX578R	15, 43	GD585R	63
D0576R	41	DX402R	9, 46	DX579R	15, 43		
D0577R	41	DX405R	9, 46	DX600R	7, 23	JF284R	19
		DX450R	9, 44	DX601R	29		
DX050R	9, 45	DX500R	5, 24	DX604R	17, 27	JG381R	5, 7, 9, 11,
DX051R	5, 45	DX501R	7, 24	DX605R	17, 27		13, 15, 48, 51
DX052R	5, 45	DX505R	35	DX606R	17, 27	JG403R	17
DX055R	9, 45	DX506T	35	DX608T	26	JG513R	28
DX094	5, 44	DX507T	35	DX610T	26		
DX100R	58	DX509R	35	DX612T	26	TE888	19
DX101R	59	DX510R	11, 32	DX614T	26		
DX102R	59	DX511R	11, 32	DX620	17, 26		
DX105R	60	DX512R	11, 32	DX621R	28		

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Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 7461 95-0 | Fax +49 7461 95-26 00 | www.aesculap.com

Aesculap – a B. Braun company

The main product mark 'Aesculap' and the product mark 'DUROGRIP' are registered marks of Aesculap AG.

Subject to technical changes. All rights reserved.
This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.

Brochure No. D11611

1011/3.0/9